

Detection of Fetal Prader-Willi/Angelman Microdeletion and Differentiation of Maternal Distal 15q Deletion by Prenatal Cell-free DNA Screening: Case Report

Background

- Prader-Willi and Angelman syndromes (PWS/AS) are rare disorders that are usually caused by genetic abnormalities called microdeletions.¹
- Some commercial laboratories now offer detection in certain microdeletion regions as part of non-invasive prenatal cell-free (cfDNA) screening.
- However, because microdeletion syndromes are rare, the available data about their detection in clinical practice settings are limited.
- **Objective:** The investigators describe a case of PWS/AS microdeletion first identified by cfDNA screening and subsequently confirmed by prenatal diagnostic testing.

Methods

- cfDNA screening using the QNatal Advanced assay, which employs massively parallel shotgun sequencing (MPSS), was conducted on a submitted blood sample.
- Maternal and fetal karyotypes were also generated, and microarray analyses were conducted to confirm suspected microdeletions.

Results

- The patient was 35 years of age and at 12 weeks' gestation at the time of cfDNA screening.
- cfDNA screening results indicated microdeletion of chromosome 15 involving the PWS/AS critical region (z score = -8.41; fetal fraction = 16%).
- Review of the karyogram (a graphical representation of sequence reads) confirmed a microdeletion within the critical PWS/AS region and also identified a suspected microdeletion involving maternal cfDNA distal to the PWS/AS region.
- Since differentiating maternal and fetal contributions to the microdeletions was difficult with cfDNA screening alone, karyotype and microarray tests were ordered.
 - Maternal and fetal karyotypes displayed normal results.
 - Maternal microarray analysis confirmed a ~2-Mb deletion of chromosome 15q13.3-q14; the deletion was categorized as a variant of uncertain clinical significance.
 - Fetal microarray analysis confirmed a ~5-Mb deletion of chromosome 15q11.2-q13.1 and the same ~2-Mb deletion as the mother.
- These results indicated a diagnosis of PWS/AS and inheritance of the maternal microdeletion.

Conclusions

- To the investigators' knowledge, this is the first reported case in which a PWS/AS microdeletion was initially detected by NIPS cfDNA screening and subsequently confirmed by prenatal diagnostic testing.
- This case demonstrates the potential of cfDNA screening and follow-up testing to identify and differentiate fetal and maternal findings, which can help guide appropriate patient management.

Poster Presented at the American College of Medical Genetics Annual Clinical Genetics Meeting

Authors

Lisa M Blazejewski, Meredith Masiello, Renius Owen, Megan D Maxwell, Carrie Guy, Ben Anderson, Ke Zhang, Neng Chen, Felicitas Lacbawan, Charles M Strom

Affiliations

Quest Diagnostics Nichols Institute, San Juan Capistrano, CA

American College of Medical Genetics Annual Clinical Genetics Meeting

Phoenix, AZ
March 23, 2017
Time: 10:00-11:30 AM

Webpage

https://acmg.expoplanner.com/index.cfm?do=expomap.sess&event_id=8&session_id=5326

Reference

1. Adams JU. *Nature Education*. 2008;1:129.